

## TC1000 Monitor Arm Adjustment

*Use this guide to adjust the Monitor Arm on the TC1000.*

### Instructions:

- **Check how easily the monitor screen tilts up and down:**
  - Gently grab the top and bottom of the monitor and see how easily it tilts up and down. If it falls down on its own, the monitor arm is too loose. If you have to put excessive force on the monitor to tilt it, then the arm is too tight.
  
- **Adjustment:**
  - Using a 4mm Flat Head Screwdriver, remove the plastic cover closest to the monitor, as shown in Figure 1.

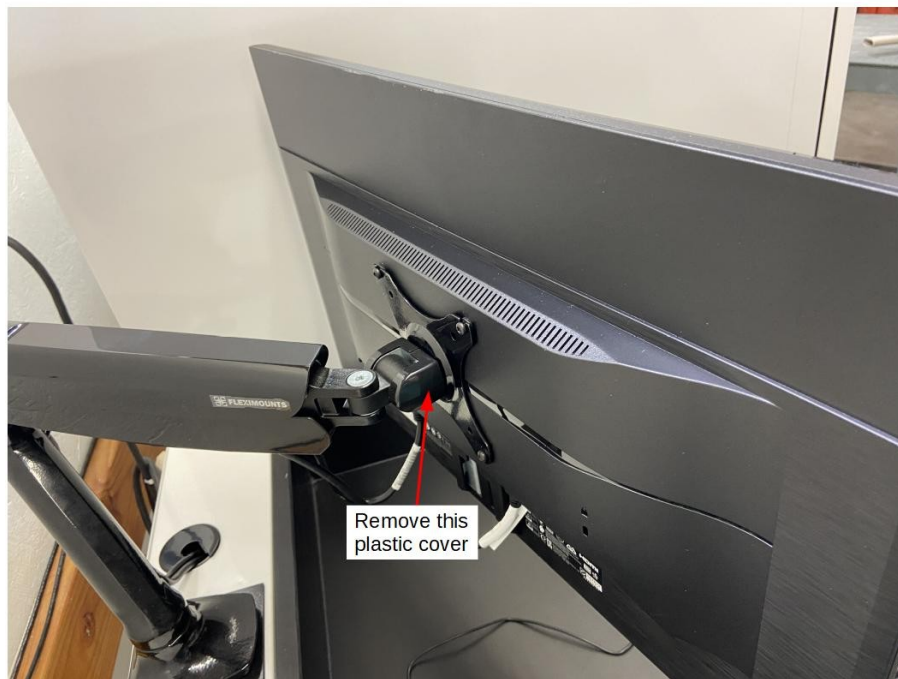


Figure 1: Cover Location

- Under the cover is the nut used for adjusting the tightness of the arm that can be adjusted using a 10mm wrench.

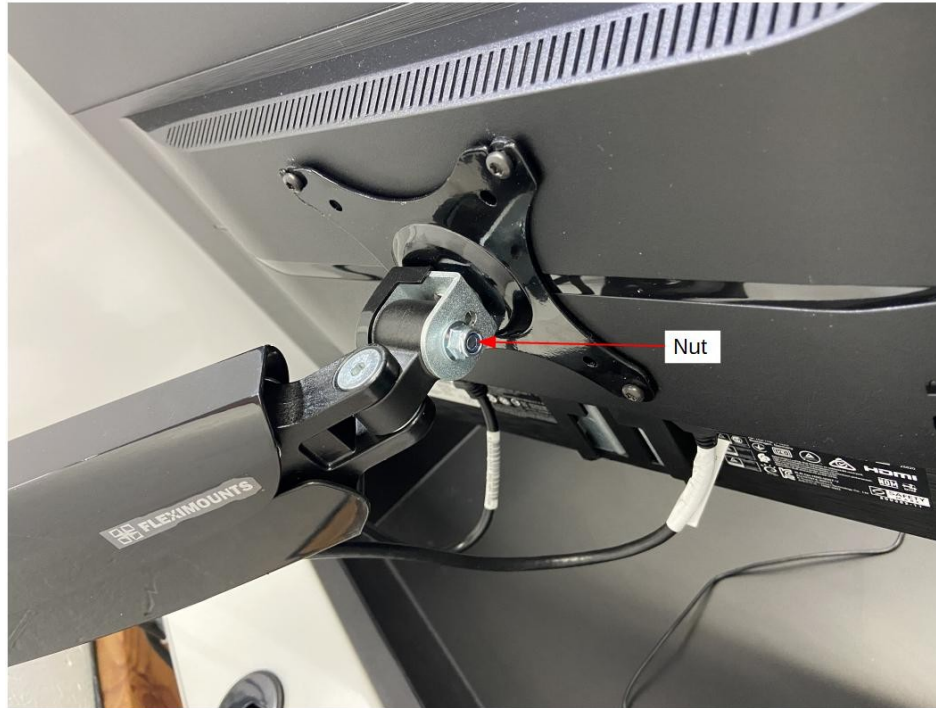


Figure 2: Adjustment Nut location

- Using a 10mm Combination Wrench, adjust the nut accordingly. Tighten the nut to tighten the arm or loosen the nut to loosen the arm.
- The monitor should not fall down from being too loose, but should not be so tight that it is difficult to move.